

# EzPAP® Positive Airway Pressure Therapy System

## Set-Up Guidelines

### 1 SET FLOW RATE


Begin at 5 LPM [fig 2]. Increase until desired PAP is reached. (Do not exceed 15 LPM.)

### 2 VERIFY PRESSURE

Should not exceed 20cm H<sub>2</sub>O [fig 1].

### 3 BREATHE

Patient should be instructed to inhale and exhale slowly [fig 3].

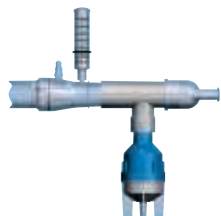

EzPAP® system  
with nebulizer

NOTE: Refer to individual product Instructions For Use for warnings, cautions, contraindications, and additional information

**Smiths Medical International Ltd**  
Hythe, Kent CT21 6JL, UK  
Tel: (0)1303 260551  
[www.smiths-medical.com](http://www.smiths-medical.com)

**Smiths Medical ASD Inc**  
1500 Eureka Park, Lower Pemberton  
Ashford Kent, TN25 4BF  
Tel: +44 (0)845 850 0445

