

SIGVARIS

# COMPREFLEX

## INELASTIC COMPRESSION WRAPS

Solutions for treatment and management of Lymphoedema and venous conditions.



- Cost effective comfortable treatment for healing of Venous Ulcer and management of Lymphoedema and Lipodema
- 14 sizes available up to 100cm calf circumference (incl. Compreflex Reduce and Strap Extension)
- Cotton liners included to protect fragile skin available in:
  - No compression
  - Compression 10-15mmHg foot & ankle only
  - Compression 10-15mmHg foot, ankle & calf
- Easy application and management of accurate compression with Accutab.

Distributed by



Order by freephone:  
Auckland: 0800 658 814  
Christchurch: 0800 804 546

Order by freefax:  
Auckland: 0800 830 660  
Christchurch: 0800 804 547

Auckland Head Office  
494 Rosebank Road, Avondale, Auckland 1026  
Email: [customerservice@uslmedical.co.nz](mailto:customerservice@uslmedical.co.nz)  
[www.uslmedical.co.nz](http://www.uslmedical.co.nz)

### 1. Choose your liner



- No compression
- Compression 10-15mmHg Foot & Ankle only
- Compression 10-15mmHg Foot, Ankle & Calf

### 2. Choose your Compreflex wrap solution




- 3 boots available with compression level 10-15mmHg & 30-40mmHg



- Calf
- Knee
- Thigh

### 3. Apply garments

### 4. Apply Accutab and adjust straps to your required level of compression

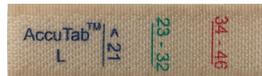

#### EASY TO USE

1. Gently lay the COMPREFLEX straps in place without stretching the fabric
2. Attach an ACCUTAB at the end of each strap
3. Pull each strap to the prescribed dosage indicated by the lines on the ACCUTAB: <21, 23-32, or 34-46mmHg

