


SORBACT® COMPRESS

Microbe binding wound contact layer

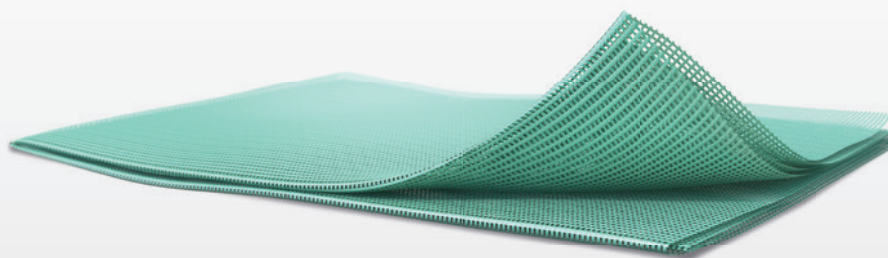

REDUCES BIOBURDEN IN SUPERFICIAL AND CAVITY WOUNDS

Sorbact® Compress is a sterile, bacteria- and fungi-binding wound dressing. It consists of a green Sorbact® wound contact layer, which allows the passage of wound exudate into a secondary dressing. The dressing is folded into eight layers. The dressing can be used in compression therapy.

Product	Ref. no.	Size	Size unfolded	Pcs/pack
Sorbact® Compress	98128	4x6 cm, 1.6x2.4 in	11x15 cm, 4x6 in	40
Sorbact® Compress	98125	7x9 cm, 2.8x3.5 in	17x28 cm, 7x11 in	40




INTENDED USE

Sorbact® Compress is intended for use in the management of clean, colonized, contaminated or infected exuding wounds, such as surgical wounds, traumatic wounds, cavity wounds, fistulas, pressure ulcers, diabetic ulcers and foot and leg ulcers.

INSTRUCTIONS FOR USE

1. Prepare the wound according to local clinical practice.
2. Select an appropriate dressing size for the wound. The dressing can be unfolded into suitable size and can overlap the wound margins if necessary.
3. Remove the dressing from the pouch using an aseptic technique.
4. If the dressing needs to be cut, use an aseptic technique. Any opened, unused dressing should be discarded.
5. Apply the dressing. Ensure that the dressing comes into direct contact with the entire wound surface. In the case of cavity wounds and fistulas, unfold and fluff up one or more dressings and line/fill the cavity or fistula. Do not pack tightly.
6. Apply an appropriate secondary dressing.
7. The dressing change frequency depends on exudate levels and the overall condition of the wound and surrounding skin. Should the clinical condition allow, the dressing can be left in place for up to 7 days.

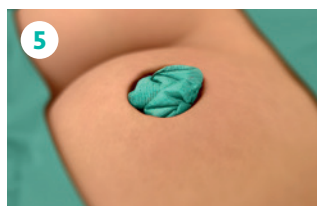

Clinicians should ensure that the product is used in accordance with the manufacturer's instructions for use.

SORBACT® RIGHT FROM THE START PREVENTS AND TREATS WOUND INFECTIONS

Sorbact® microbe binding dressings prevent and treat wound infections by lowering the bioburden. Microbes bind to the dressing surface and are removed when the dressing is changed.

Sorbact® is a Swedish innovation and the range of wound dressings is available through ABIGO partners and distributors in more than 65 countries worldwide. In some regions, Sorbact® is available as Cutimed® Sorbact® and Leukomed® Sorbact®. Find your local distributor at abigo.com. Sorbact® is a registered trademark of ABIGO Medical AB.