

## Cleaning of Lithium Ion Handles from HillRom's Sheet

The use of cleaners/disinfectants which contain quaternary ammonium have demonstrated negative effects on plastics used in these products, causing them to crack.

- Compatible with:

Wipe the handle and charger with an appropriate health care low- or intermediate level cleaner/disinfecting wipe that incorporates either a **1:10 sodium hypochlorite (bleach) solution** or **isopropyl alcohol** as the active disinfection ingredient.

- Not compatible with: quaternary ammonium based cleaner disinfectants.

These quaternary ammonium based wipes are made by several companies; the more popular brands are PDI Sani-Cloth (Plus, Super, AF & AFIII), Metrex CaviWipes, and Ecolab DisCide.