

SURGI-GEL PLUS™ STERILE SACHETS 3mL

Composition	A viscous, water based, sterile adhering gel. Surgi-gel PLUS™ Sterile sachets contain purified water, glycerol, carbomer 940, propyl hydroxybenzoate and methyl hydroxybenzoate as preservatives. Sterilised by gamma irradiation.
Use	Surgi-gel PLUS™ Sterile sachets contain a sterile, greaseless, transparent, water-based, non-staining, non-irritating lubricating gel. Recommended for specific surgical and gynaecological lubrication when increased adherence is needed.
Contraindications	Known hypersensitivity to methyl hydroxybenzoate and propyl hydroxybenzoate.
Directions for use	Apply to instruments or directly to mucosal surfaces and orifices as required. Single use only.
Adverse Reactions	Surgi-gel PLUS™ Sterile sachets are normally well tolerated, but should skin irritation occur, discontinue use. Sensitisation has occurred when medications containing hydroxybenzoates have been applied to damaged or broken skin. After topical use, local irritation may occur. Hypersensitivity reactions have been reported.
Storage	Store in a cool dry place out of direct sunlight.
Poison Schedule	Not scheduled using the criteria in the Standard Uniform Schedule for Medicines and Poisons.

Presentation

Code	Product	Size
SUR03038F	SURGI-GEL PLUS™ STERILE SACHETS	3mL

Contact Point

Medical Information Associate
Distributed by Perrigo Australia
Orion Laboratories Pty. Ltd., trading as Perrigo Australia
ABN 56 009 293 136
25-29 Delawney Street, Balcatta, Western Australia 6021 AUSTRALIA
Telephone (all hours): +618 9441 7800
FREE PHONE: 1800 805 546 FREE FAX: 1800 004 110
EMAIL: customerservice@perrigo.com.au; WEBSITE: www.perrigo.com.au
ORION® is a registered trademark of Orion Laboratories Pty Ltd

END OF USER GUIDE