

9100 Ray Lawson Anjou, Quebec, Canada P.O Box 448 Champlain, NY 12919 USA Tel (514) 356 1224 Fax (514) 356 0055

Toll free: (800) 361-4964

General Laundering Instructions

DO NOT OVERBLEACH!

- Too much bleach may cause premature barrier deterioration and degradation of the cotton and soaker material. DO NOT use more than 1.5 quarts (1.4 l) of 1% chlorine bleach per 100 lbs (45kg) 3/4 quart (0.7 l) of 1% chlorine bleach per 50 lbs (22 kg)
- IMPORTANT: Addition of ANTI-CHLOR at 1st or 2nd rinse AFTER chlorine bleach operation will help eliminate residual bleach or bleach carryover.
- Use of oxygenated bleach is recommended.
- Do not pre-soak products in bleach.

DO NOT OVERLOAD!

- Load equipment to 80% soiled weight.

SOFTENER NOT RECOMMENDED!

- Softener will reduce absorbency of all incontinence care products.
- We recommend not using softener or using it in a minimal amount.
- For all "Chamonix" & "Sonoma" products DO NOT USE SOFTENER

DO NOT OVERDRY!

- Excessive drying temperature may cause deterioration and excessive shrinkage of fabrics.
- Set drying temperature at 65 C (150 F), maximum 75 C (170 F). 3% to 5% humidity should remain in the product after drying. Drying time may vary according to the type and capacity of the equipment.
- Allow a cool down of at least 5 to 10 minutes after drying cycle is completed to control wrinkles and increase fabric life.
- IMPORTANT DO NOT leave goods in dryer or hamper! As soon as they are cool, fold and store. As with any items, spontaneous combustion can occur if improperly handled.

NOTES

- Steam should NEVER be applied directly to the wash wheel.
- Final pH should be from 5.5 to 6.0.
- Quaternary bacteriostat added to the sour cycle helps to inhibit longterm bacterial growth.
- Formulas, chemical amounts, pH and titrates will vary depending upon chemical supplier, local water conditions and equipment. Contact your chemical supplier for further information.

Special Product Recommendations

Chamonix & Sonoma Products

Do Not Use Softener. Softener will reduce or prevent surface absorbency. **- Do not overdry**. Use a full cooldown of at least 5 to 10 minutes and remove from dryer immediately. **-** Excessive heat can cause surface damage.

PERMA PRINT Adult Bibs/Clothing Protectors

Do not use softener. We recommend using a medium soil formula. Chlorine bleach may be used.

Impermeable Laundry Bags

- When used with incontinence care products, launder in the same formula used for MIP incontinence care products.
- Dry for approximately TEN MINUTES! Do NOT leave in dryer at end of cycle.
- Bags may also hang dry.
- Typical drying schedule in a high capacity (300-400 lb) gas drier: 10 min at 160oF (67oC)

Nylon Laundry Bags

- wash nylon bags SEPARATELY
- prewash bags before putting them into service
- Wash in a LIGHT formula. Chlorine Bleach not recommended May remove color.
- Do not overdry

SLEEP-KNIT PLUS, ULTRA-KNIT and DUETTE

- Recommended laundering instructions are available separately.
- For more information please contact your MIP consultant.

To Repair Satinlite barrier, contact

THERMOPATCH,

Barrier Patch # 820 (Green), #821 (White) Canada: 800-265-6416 U.S.: 800-252-6555

International Accounts: Contact MIP Head Office

Date Created: 01/21/97 11:20:40 AM Date Changed: 01/18/2007