

Garment Measuring Guide


The following guide is intended as an aid to help the user select the correct size garment .

The treatment type will determine the garment type e.g. **hydroven® 1** for uniform compression, **hydroven® 3** for graduated pressure or **hydroven® 12** for specialist multi chamber therapy.

When determining the correct size garment:

- Remove any bandaging, stockings or bulky dressings
- Measure the points as shown on the diagram below
- DO NOT pull the measuring tape tight
- Make sure the garment extends beyond the area under treatment

Refer to the garment list at the end of this document.

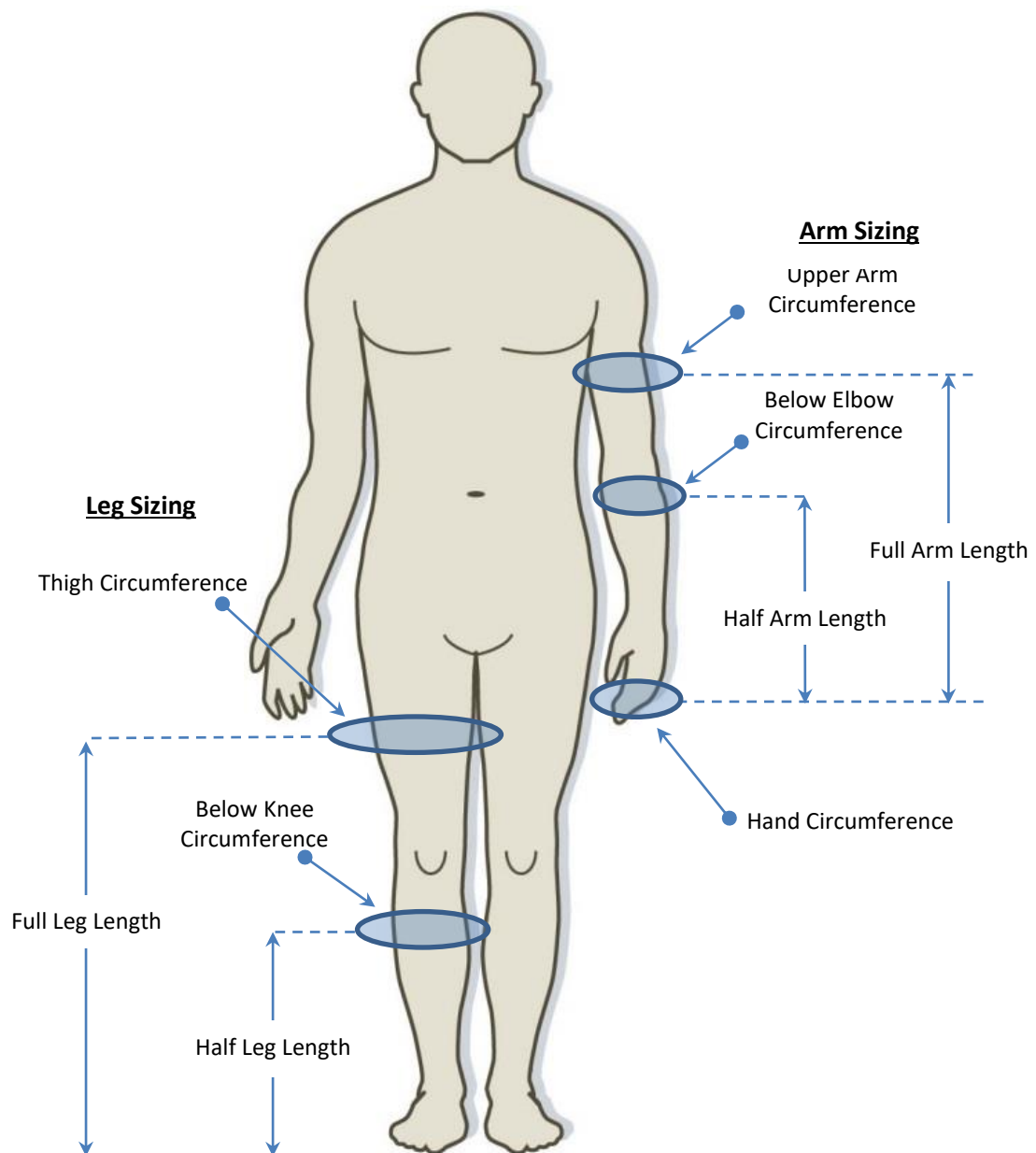

Hydroven - 1 SINGLE Chamber Leg Garments			
Order Code	Type	Length (L)	Thigh Circumference (W)
5101L50	Half Leg	50cm	61cm
5101L66	Full Leg	66cm	64cm
5101L71	Full Leg	71cm	66cm
5101L76	Full Leg	76cm	72cm
5101L84	Full Leg	84cm	72cm
5101L92	Full Leg	92cm	72cm

Hydroven - 3 THREE Chamber Leg Garments			
Order Code	Type	Length (L)	Thigh Circumference (W)
5103L50	Half Leg	50cm	61cm
5103L66	Full Leg	66cm	64cm
5103L71	Full Leg	71cm	66cm
5103L76	Full Leg	76cm	72cm
5103L84	Full Leg	84cm	72cm
5103L92	Full Leg	92cm	72cm

Hydroven - 1 SINGLE Chamber ARM Garments				
Order Code	Type	Length (L)	Circumference Hand (H)	Circumference Shoulder (S)
5101A51	Half Arm	51cm	44cm	56cm
5101A68	Full Arm	68cm	44cm	62cm
5101A78	Full Arm	78cm	44cm	62cm

Hydroven - 3 THREE Chamber ARM Garments				
Order Code	Type	Length (L)	Circumference Hand (H)	Circumference Upper Arm (S)
5103A68	Full Arm	68cm	44cm	62cm
5103A78	Full Arm	78cm	44cm	62cm

Hydroven - INSERTS (Compatible with Hydroven 1 and Hydroven 3 garments)				
Order Code	Type	Length (L)	Added Circumference Wide End	Added Circumference Narrow End
510LI50	Half Leg Insert	50cm	19cm	14cm
510LI66	Full Leg Insert	66cm	19cm	14cm
510LI71	Full Leg Insert	71cm	19cm	14cm
510LI76	Full Leg Insert	76cm	19cm	14cm
510LI84	Full Leg Insert	84cm	19cm	14cm
510LI92	Full Leg Insert	92cm	19cm	14cm
510AI68	Full Arm Insert	68cm	17cm	12cm
510AI78	Full Arm Insert	78cm	17cm	12cm

Hydroven 12 - TWELVE Channel Arm Garments			
Order Code	Type	Length (L)	Circumference Upper Arm (S)
316A68	Full Arm	68cm	62cm
316A78	Full Arm	78cm	62cm
Hydroven - 12 TWELVE Channel Arm Inserts			
Order Code	Type	Length (L)	Additional Width at Upper Arm
316AI68	Full Arm	68cm	17cm
316AI78	Full Arm	78cm	17cm

Hydroven 12 - TWELVE Channel Leg Garments			
Order Code	Type	Length (L)	Thigh Circumference
316L76S	Standard Leg	76cm	71cm
316L84S	Standard Leg	84cm	71cm
316L76W	Wide Leg	76cm	79cm
316L84W	Wide Leg	84cm	79cm
Hydroven - 12 TWELVE Channel Leg Inserts			
Order Code	Type	Length (L)	Additional Width at Thigh
316LI76	Leg Insert	76cm	19cm
316LI84	Leg Insert	84cm	19cm