

Maxxcare®

Heel Pro Evolution



**Prevents and treats
heel pressure injuries**

Heel Pro Evolution

Purpose


Is used to offload pressure on the heel and ankle.
For effective intervention to prevent or treat existing heel pressure injuries.

Use

Single patient reusable. For ward use. The client may take home.
Easy to clean and also machine washable for durability and for ongoing therapy and treatment.

Quick and easy to fit

1. Measure for the right size
2. Write the client name on the label
3. Open the velcro straps
4. Place the patients foot into the Heel Pro Evolution, loosely fasten the velcro straps.

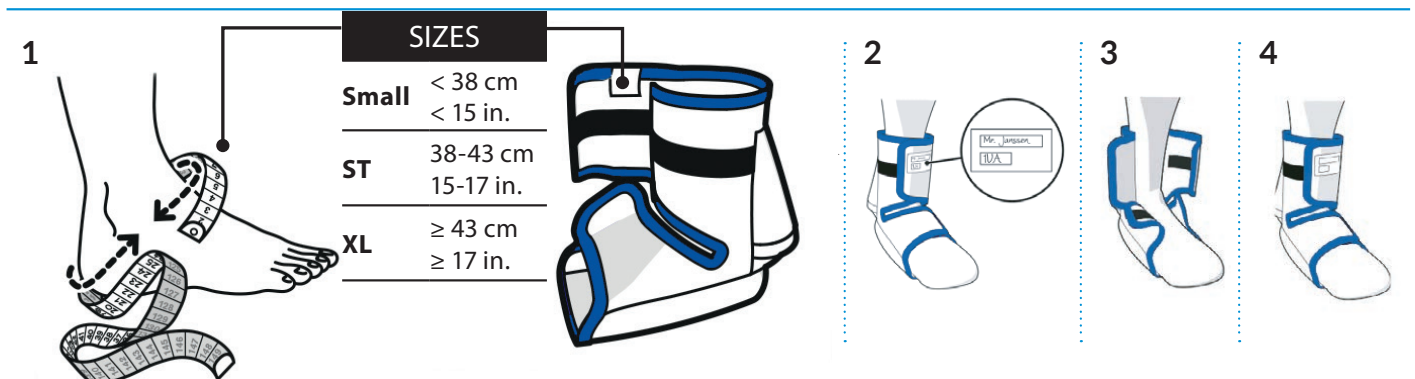

Please refer to the Maxxcare User Manual
for full purpose, use, precautions, safety instructions, maintenance and warranty.

USL CODE	DESCRIPTION	UOM
060004	Maxxcare Heel Pro Evolution Small	EACH
060001	Maxxcare Heel Pro Evolution Standard	EACH
060003	Maxxcare Heel Pro Evolution XL	EACH