

Lastodur®

the permanently elastic long-stretch
bandages for durable compression bandaging



Lastodur® – a long-stretch compression bandage

Their high extensibility of approximately 170-180% makes Lastodur permanently elastic long-stretch bandages very versatile. They are available in two versions: Lastodur strong for high compression and Lastodur light for lighter compression. Lastodur strong is also available as a joint bandage in an indication-related shorter version with a length of 3.5 m.

Lastodur provides even, sustained pressure, which acts on the superficial blood vessels.

Lastodur is highly conformable and produces a very secure finished bandage.

Lastodur is permeable to air, kind to the skin and durable.

The bandages may be washed at 60°C and sterilised (by autoclaving at 134°C).

Its permanent elasticity makes Lastodur re-usable and cost-effective.



Lastodur® light

Permanently elastic long-stretch bandage, elasticity of approximately 180%. Tension can be varied to provide controlled compression, highly conformable, wear resistant; permeable to air and kind to the skin, durable, may be washed at 60°C and sterilised (by autoclaving at 134°C).

- Constructed from:
 - 84% viscose
 - 11.5% polyamide
 - 4% polyurethane
 - 0.5% cotton
- compression level: 22 mmHg*
- suitable as layer three in a 4-layer bandaging system

Contraindication:

- patients with severe arterial disease



Lastodur light for moderate to low compression

Lastodur® strong

Permanently elastic long-stretch bandage, elasticity of approximately 170%. Tension can be varied to provide controlled compression, highly conformable, wear-resistant; permeable to air, kind to the skin and durable, may be washed at 60°C and sterilised (by autoclaving at 134°C).

- Constructed from:
 - 85% cotton
 - 8% polyurethane
 - 7% polyamide
- compression level: 38 mmHg*
- skin-coloured

Contraindications:

- patients with arterial disease
- fixed ankle



Lastodur strong for high compression

* Bandages applied with 50% extensibility (load 10 N/cm), with 50% overlap on an ankle circumference of 25 cm

Benefits

- Latex-free
- Cotton base is breathable for wearer comfort
- Lastodur light has a central coloured thread which allows easy application



	Width	Unit	Code No.	Units Per case
Lastodur® light With bandage clips, Stretched length 8.7 m Individually boxed	10 cm	1 bandage	931 652	60
Lastodur® strong With bandage clips, Stretched length 8.7 m Individually boxed	6 cm	1 bandage	931 663	60
	8 cm	1 bandage	931 664	60
	10 cm	1 bandage	931 665	60
	12 cm	1 bandage	931 666	30
	20 cm	1 bandage	931 668	20

www.hartmann.com.au

THE HARTMANN ALLIANCE

NEW SOUTH WALES	VICTORIA & TASMANIA	QUEENSLAND	SOUTH AUSTRALIA & NT	WESTERN AUSTRALIA	NEW ZEALAND
PAUL HARTMANN Pty Ltd Phone: 1800 805 839	Sutherland Medical Pty Ltd Phone: 1300 664 027	Medical & Surgical Requisites Pty Ltd Phone: 1800 801 226	McNeil's Surgical Pty Ltd Phone: 08 8363 0888	PAUL HARTMANN Pty Ltd Phone: 08 9344 3111	USL Medical Ltd Phone: 0800 658 814