

Product Description

Solo RepoSheet has been specially developed by Liko to facilitate transfers of a patient in bed. It is an excellent aid for repositioning further up in bed and during turning in beds using a lift and with minimal body strain for the caregivers.

Transfers can be carried out with the help of one person but we recommend that there are two caregivers which is time-saving and a comfortable way to work.

Solo RepoSheet can be used in combination with, or instead of ordinary bed sheets.

Solo RepoSheet is intended for short-term use by one individual patient. It should be disposed when it becomes soiled or when the patient no longer needs it.

There are also RepoSheet models for long-term use in Liko's product range.

△ If it is necessary to transfer a patient between two horizontal surfaces sufficient lift straps are used to ensure that the patient has enough support under the body to be lifted safely during the transfer. Also, make sure that strap loops do not become stuck anywhere during the transfer.

Solo RepoSheet works best in combination with overhead lifts. We recommend Likorall, Multirall or Liko MasterLift. See final page for recommended slingbars.

△ Solo RepoSheet is **not** intended for lifting or transfers from lying to sitting positions. For such needs, we recommend lifting with one of our many sling models.

In this document, the person being lifted is referred to as the patient and the person helping them is referred to as the caregiver.


△ is a warning triangle used for situations that demand extra care and attention.

IMPORTANT!

Lifting and transferring a patient always involves a certain level of risk. Read the instruction guide for both the patient lift and lifting accessories before use. It is important to completely understand the contents of the instruction guide. The equipment should only be used by trained personnel. Ensure that the lifting accessories are suitable for the lift used. Exercise care and caution during use. As a caregiver, you are always responsible for the patient's safety. You must be aware of the patient's ability to make it through the lifting situation. If something is unclear, contact the manufacturer or supplier. Instruction guides can be downloaded free of charge at www.liko.com.

User Description

Definitions

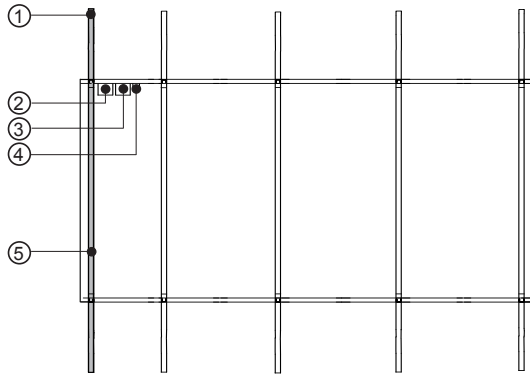


- | | |
|-------------------------------|---|
| 1. Strap loops | 4. Year/month of production |
| 2. Product label | 5. Lift strap (with different colour at the head end) |
| 3. Caution label: Do not wash | |



△ Before lifting, remember the following!

- Plan the lifting operation before the lift to make it as safe and convenient as possible.
- Although Liko's slingbars are equipped with safety latches, special caution must be exercised: Before the patient is lifted from the underlying surface, but after the straps have been fully extended, make sure the strap loops are properly connected to the slingbar.
- Never leave a patient unattended during a lifting situation.
- The wheels on the bed must be locked during lifting/transfer operation, the recommendation applies in particular when turning patients.
- Put the bed rails up during turning, particularly if only one caregiver carries out the procedure.
- Make sure that the RepoSheet's unconnected strap loops do not become stuck, for example, in the bed during use.
- During transfers, the caregivers should hold on to the slingbar and/or support the patient from underneath. Avoid pulling the lift straps as that can damage the RepoSheet.

Positioning of Solo RepoSheet


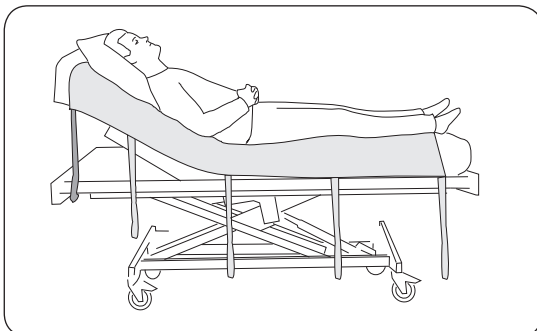


Place Solo RepoSheet centrally on the bed with the head end of the sheet approx. 10 cm (3.9 inch.) from the edge of the mattress. Note that the lift strap at the head end has a different colour. Solo RepoSheet should be placed like this the first time it is used or when it is time to change the bedding.

The first time Solo RepoSheet is put in place on the bed, it is easiest to lift the patient from the bed in a sling, for example, Soft Original High Back Sling, Model 26.

Each new time Solo RepoSheet is put in place means repositioning 2-3 times further up in the bed before it is time to change the bedding.

Lifting further up in the bed

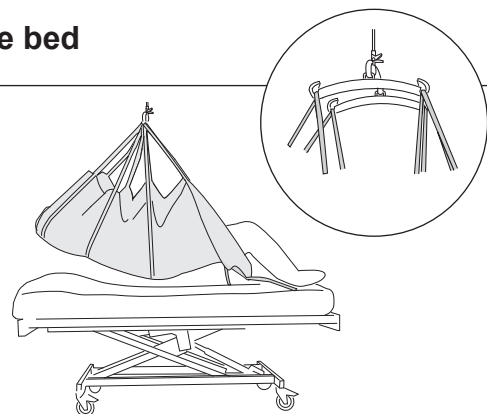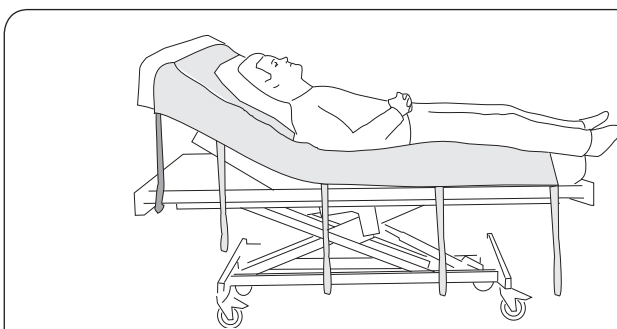

Connect the strap loops: Connect the strap loops to each side of the slingbar. Use the lift straps that lift the head and all the lift straps that lift the body down to the hollow of the knees (3-4 pairs of straps is usual). △ Lifting with the lift straps under the hollow of the knees can cause over-extension of the knee-joint. When it is also necessary to lift the calves, (for example when transferring between two beds) particular care must be taken so that over-extension of the knee-joints does not occur.

Lifting: Lift so that the patient is free of the mattress and can be moved up to the desired position in the bed. During repositioning, the caregivers should hold on to the slingbar and/or support the patient from the underneath. **Do not pull the lift straps!** Lower, disconnect and smooth out the Solo RepoSheet.


Alternative: In some cases, particularly if you want to lift the patient in as a straight position as possible, it may be better to use Side Bars and distribute the strap loops on the Side Bar's hooks instead, see the small illustration (although not during the "turning" operation when only one side of the Solo RepoSheet's lift straps are carrying weight).

Turning in bed

Turning a patient in bed can be done with the assistance of one person but we recommend that there are two caregivers which is time saving and a comfortable way to work.

△ Use of bed rails is recommended, particularly in situations where only one caregiver is carrying out the procedure.


Preparations for turning:

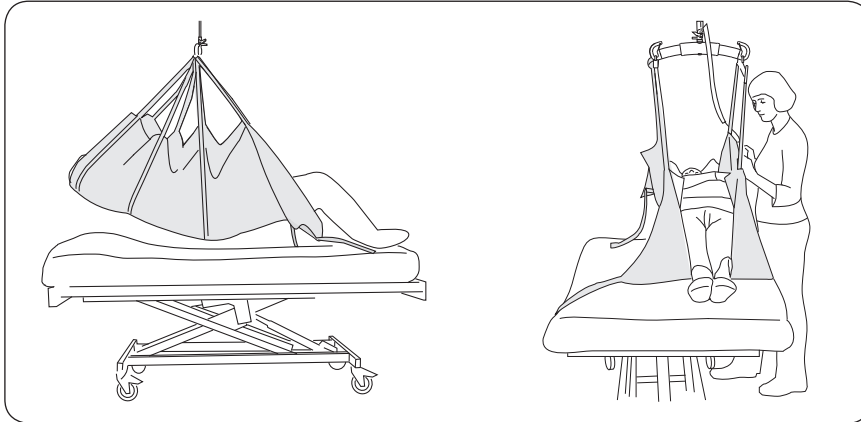


1. Before turning can take place the patient must first be moved to one side of the bed. Connect the necessary number of lift straps according to the description at the end of page 2. Lift the patient high enough to clear the bed surface.

2. Move the patient to one side of the bed, lower onto the mattress.

Turning:

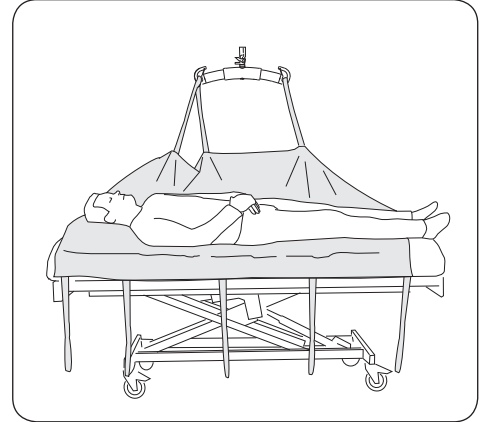

1. Disconnect the strap loops from the side of the slingbar which is closest to the middle of the bed. If Side Bars are used, these should be removed. Connect 3-4 strap loops to the hooks on the slingbar when it is hanging parallel to the bed.

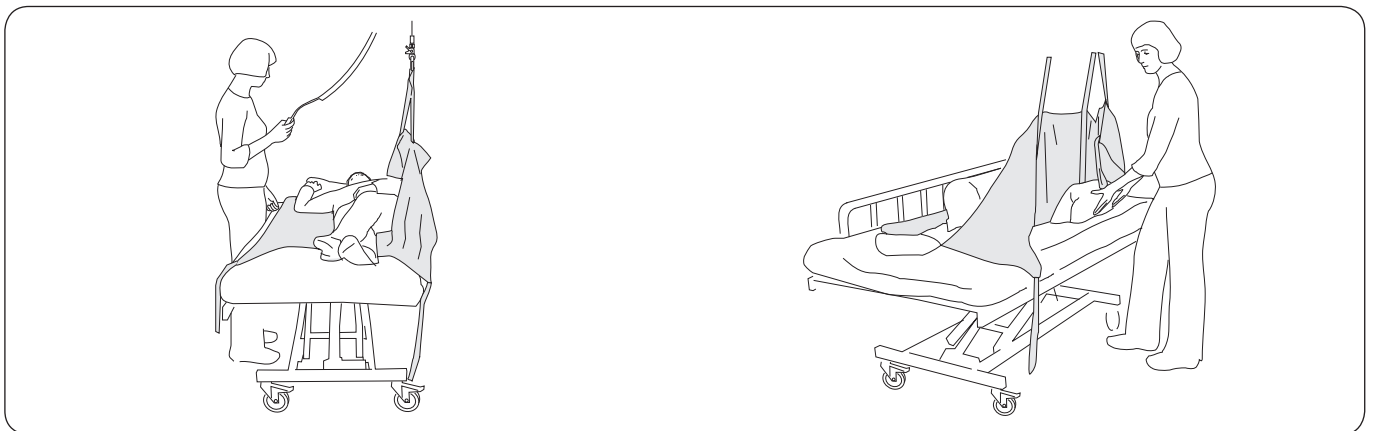

2. Place the patient in such a way that turning is made easy: arms over the side, one leg over the other. △ Lift carefully so that the patient is turned to the side with the help of the lift without tipping forward.

3. Adjust the patient's body position to one which is comfortable and place pillows to support (behind the back, between the knees, under the head). Lower the slingbar and disconnect the strap loops. Smooth out the sheet.

When the patient is to be moved from the side position onto a supine position carry out the procedure in reverse order: remove pillows, lower lift, connect the strap loops, lift and move the patient to the middle of the bed, etc.

Changing Solo RepoSheet

The easiest way to change Solo RepoSheet is to lift the patient in a RepoSheet while fitting the bed with a new one. After lowering, the first sheet can be removed when the patient is turned to the side, preferable with the assistance of the new RepoSheet and the lift.

If Solo RepoSheet is used for lifting further up in the bed it is practical to have two or more sheets to change between.

Overview, Solo RepoSheet

Name	Model	Prod. No.	Maximum load
Solo RepoSheet	Short	36871003	200 kg (440 lbs.)

Material: 50 % cotton and 50 % polyester

Accessories: Universal SideBars, pair. Prod. No. 3156079

Care and Maintenance

△ Solo RepoSheet is a disposable item that is intended for individual use only. Write the patient's name on the product label which is on the back side of the sheet.

Solo RepoSheet should no longer be used:

- when it is soiled or if it is suspected of being contaminated
- if it is damaged
- when the patient no longer requires it.

Dispose of the Solo RepoSheet accordance with hospitals or care facility regulations. Solo RepoSheet can be recycled by incinerating.



DO NOT WASH. The caution label situated on the back of the sheet displays the DO NOT WASH –symbol. If the sheet is washed this label disintegrates to display the DO NOT USE –symbol. The sheet must then be disposed of.



Inspect the Solo RepoSheet before each use. Pay special attention to wear and damage to seams, material/fabric, straps and strap loops. Do not use damaged lifting accessories. If anything is unclear, please contact the manufacturer or supplier.

Recommended combinations of Liko's slingbars:

	Universal 350	Slim 350	Universal 450	Standard 450	Universal 600
Lifting and positioning further up in bed:					
Solo RepoSheet Short	1	1	1	1	1

	Universal 350	Slim 350	Universal 450	Standard 450	Universal 600*
Lifting and positioning when turning in bed:					
Solo RepoSheet Short	1	1	1	1	4

Explanation: 1 = Recommended, 2 = May work, 3 = Not advised, 4 = Inappropriate

All recommendations apply both with and without SideBars.

* We recommend slingbars with a maximum width of 450 mm (17.7 inch.). A slingbar which is too wide can damage the Solo RepoSheet.

CE Medical technical class I product

Liko's products undergo continuous development, which is why we reserve the right to make product changes without prior notice. Contact your Liko representative for advice and information about product upgrades.

Design and Quality by Liko in Sweden

Liko is quality certified according to ISO 9001 and its equivalence for the medical device industry, ISO 13485. Liko is also certified according to environmental standard ISO 14001.



A **Hill-Rom** Company

www.liko.com

Manufacturer:

Liko AB
SE-975 92 Luleå
Sweden
info@liko.se

National Distributor in USA and Canada:

Liko
122 Grove Street
Franklin, MA 02038
Tel: 508-553-3993, 888-545-6671
Fax: 508-528-6642
info@liko.com

Liko Distributor in UK:

Liko (UK) Ltd. • Brunel Centre,
Stroudwater Business Park,
Stonehouse, Gloucester
GL10 3RU, United Kingdom
Phone: +44 (0) 1453 827272
Fax: +44 (0) 1453 828844
info@liko.co.uk

