
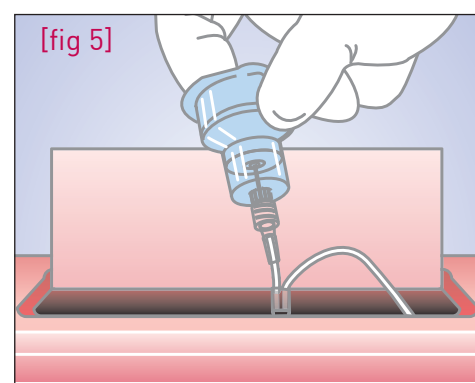
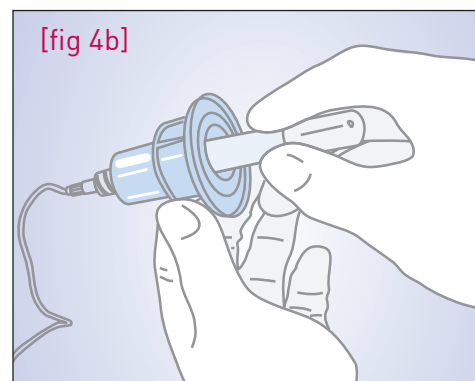
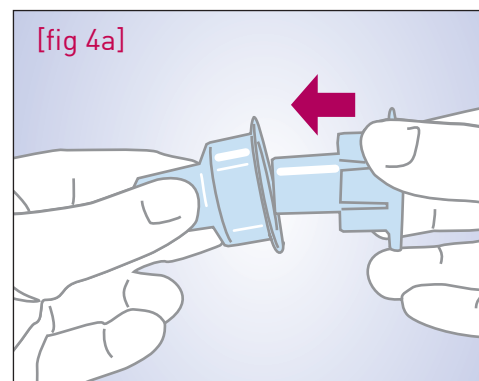
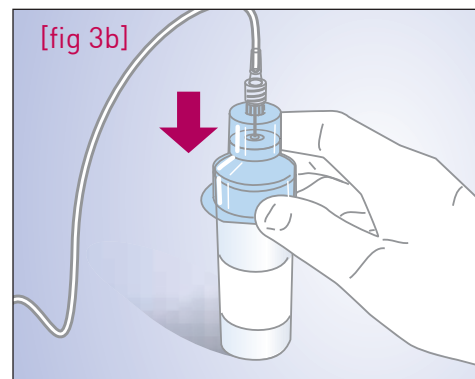
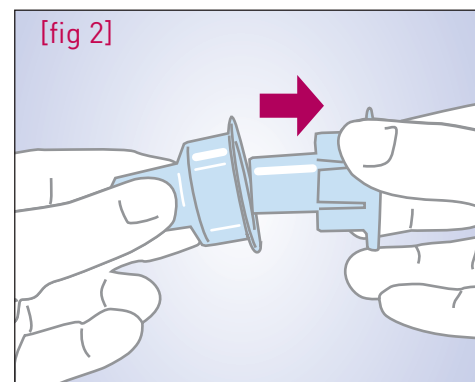


Saf-T Holder® Blood Culture Device with Male Luer

Step-by-Step Safety Guidelines

IMPORTANT: Please refer to the Instructions for Use supplied with the product for detailed instructions, warnings and cautions.

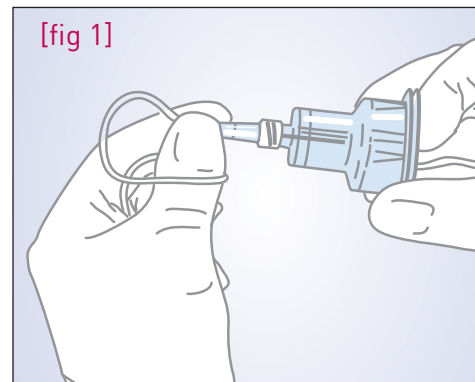

Using Saf-T Holder® Blood Culture Device with Saf-T Wing® Blood Collection Set or Equivalent

1 Attach Saf-T Holder® device

- Peel open the blister pouch and remove the device.
- Firmly secure the male Luer of the Saf-T Holder® Blood Culture Device to female Luer of Saf-T Wing® Blood Collection Set or equivalent [fig 1].

2 Remove vacuum tube adapter

- Remove vacuum tube adapter from Saf-T Holder® device [fig 2].
- Perform venipuncture per institution policy.

3 Collect blood sample

- Place blood culture bottle on a flat surface with the access point facing up [fig 3a].
- Collect blood sample by placing Saf-T Holder® device over blood culture bottle and pressing down to penetrate the blood culture bottle septum. Hold the blood culture device down onto the blood culture bottle during collection [fig 3b].

Using Saf-T Holder® Blood Culture Device with a Vacuum Tube Adapter and Saf-T Wing® Blood Collection Set or Equivalent

4 Collect blood into vacuum tube

- When additional blood is required for other tests, insert the vacuum tube adapter into the blood culture bottle holder until a "click" is felt or heard [fig 4a]. The blood culture bottle holder is now ready to be used with standard blood collection vacuum tubes [fig 4b].
- Perform blood draw according to institution policy. Additional vacuum tubes may be used as appropriate.

- 5. After use, dispose of sharps and contaminated product into an approved sharps container [fig 5].

Jelco® – Generations of Safety