

# INNOV8<sup>✓</sup> Low



## Innov8 Low Hospital Bed

Contact our Customer Service Team for more information  
Tel: +44 1422 233 000 / E: [sales@sidhil.com](mailto:sales@sidhil.com) / [www.sidhil.com](http://www.sidhil.com)

...making it better

## Innov8 Low New Hospital bed from Sidhil

The Innov8 Low height bed has an exceptional low height and with its key features, can be utilised as an everyday acute ward bed without compromising on patient outcomes.

Falls prevention is a major risk management consideration in hospitals today. Sidhil's Innov8 Low acute hospital ward bed provides a fully functional solution, with the added benefit of an extremely low platform height at just 22.5cm.

From this minimum level, the Innov8 Low bed rises to 71cm and incorporates a comprehensive range of electrically operated functions including auto contouring, cardiac chair function and auto regression to avoid surface pinching and patient sliding. With both Trendelenberg and reverse Trendelenberg positioning, the Innov8 Low is capable of handling maximum patient weights of up to 30 stone (190kgs).


Platform panels incorporate specially designed ridges for breathability, to simplify decontamination and to improve infection control. All beds in the Innov8 range are reliable and easy to maintain, incorporating superb ergonomics in terms of both manual handling and user comfort.

...making it better

[www.sidhil.com](http://www.sidhil.com)

### Cantilever Side Rails

The Innov8 Low is supplied without siderails. Standard or high removable cantilever style siderails are available as an optional extra.

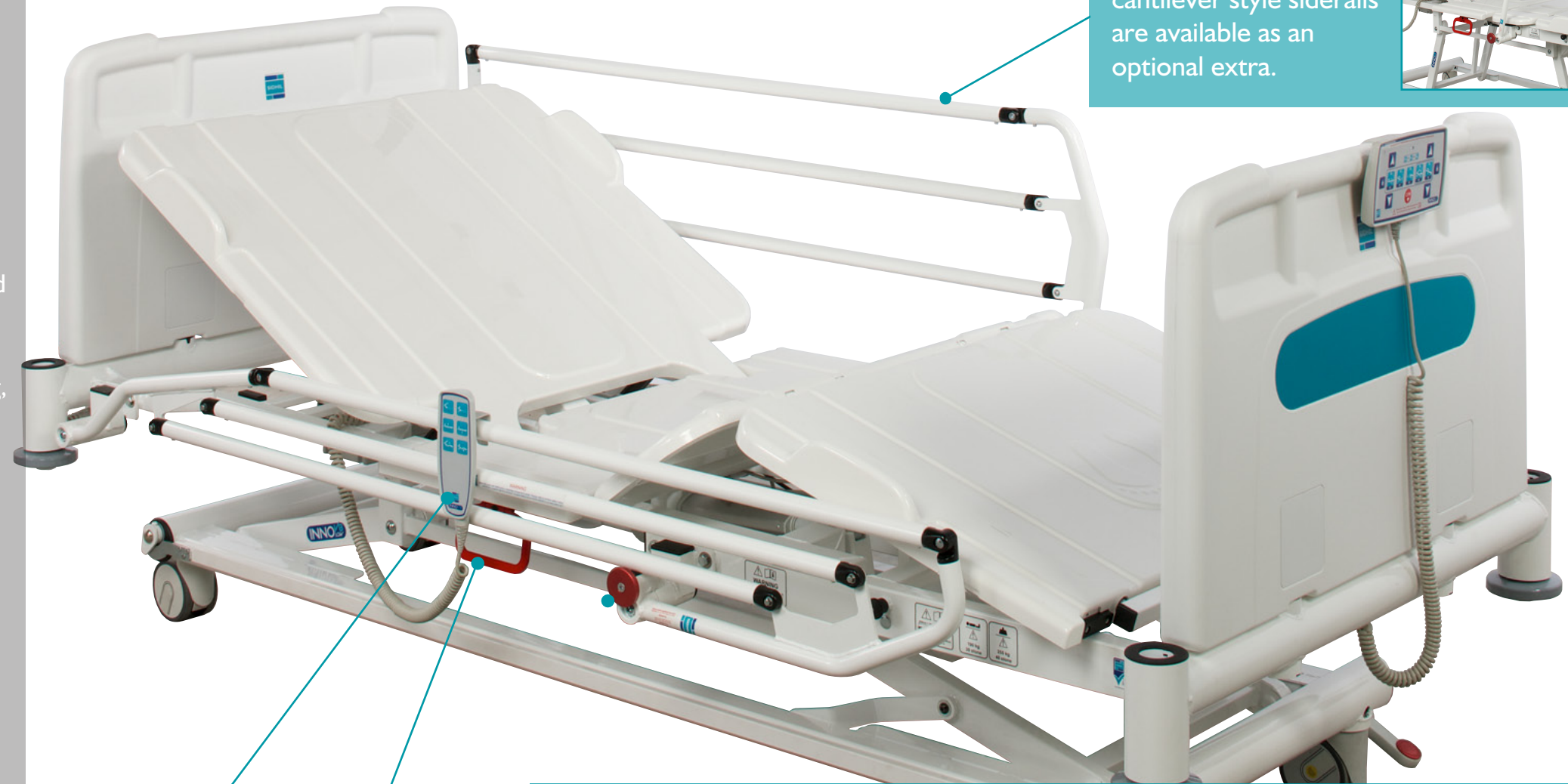

### Control Handsets

Simple to use Patient Handset and Nurse Control Panel, with handset lockout.




### CPR function

The dual sided CPR levers flatten both the backrest and the knee break to make the bed a usable height in the event of a cardiac emergency.



### Lowering the Bed

When lowering the bed from maximum height the bed will travel at full speed until it reaches 42cm, at which point it will slow and a short warning alarm will sound to alert the carer that the bed is now travelling lower than a standard bed. The bed can only be positioned at its lowest height with the Nurse Control Panel.

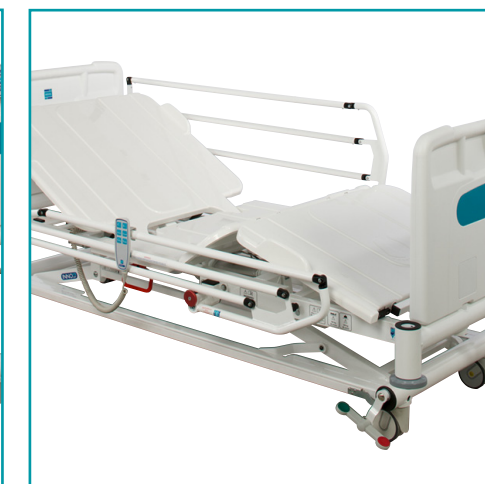

# Innov8 Low

## Key Features :



- Exceptional low-to-high range 22.5cm to 71cm
- Maximum patient weight of up to 30 Stone (190kg)
- 3 year warranty (1st year parts & labour, 2nd year parts only)
- The Innov8 Night Pack illuminates the underside of the bed improving patient safety and security
- Easy to clean and decontaminate, incorporates BioCote®
- Electrically operated backrest, height & knee break with battery backup
- 12cm under bed clearance for hoist (minimum mattress platform height 40cm)
- Auto-cardiac chair position (as per standard Innov8)
- Central locking braking system - dual sided
- Extendable mattress platform length for taller patients (+ 20cm)
- Easy to manoeuvre, reducing back or handling problems
- Removable head/foot ends with colour coding options
- IP54 standard ingress protection
- **UK manufacture and support**



## Innov8 Low Product Code INNOV8/LOW/AQ

### General Accessories :

Innov8 Lifting pole <b>INNOV8/LP</b>	Side Rail Pad (standard side rail) <b>I316/PS/CREAM</b>
Innov8 IV pole <b>INNOV8/DP</b>	Side Rail Pad (high side rail) <b>I316/PS/UNIVERSAL</b>
Innov8 Traction – Balkan Beam <b>INNOV8/TBB</b>	Crash Mattress <b>MAT/FALL</b>
Innov8 Traction – Foot End <b>INNOV8/TFE</b>	

A full range of mattresses are available from Sidhil for the Innov8 Range

### Factory Fitted Accessories :

Innov8 Cantilever side rails (standard) <b>INNOV8/LOW/SR</b>
Innov8 Cantilever side rails (high) <b>INNOV8/LOW/HSR</b>
Innov8 Night pack <b>INNOV8/LOW/NP</b>
Innov8 Bed End <b>INNOV8/BE/(Colour)</b>

### Dimensions :

Overall Length	<b>218cm (238cm when fully extended)</b>
Overall Width	<b>102cm</b>
Mattress Platform Height (minimum)	<b>22.5cm</b>
Mattress Platform Height (maximum)	<b>71cm</b>
Height Overall (minimum)	<b>72cm</b>
Height Overall (maximum)	<b>120cm</b>
<b>Maximum Patient Weight</b>	<b>190kg (30st)</b>
<b>Maximum Safe Working Load</b>	<b>255kg (40st)</b>



Paper from  
Responsible sources  
[www.fsc.org](http://www.fsc.org)



Certificate No. FM14550

Sidhil Ltd, Sidhil Business Park  
Holmfield, Halifax  
West Yorkshire, HX2 9TN



**Tel:** +44 (0) 1422 233 000 / **Fax:** +44 (0) 1422 233 010  
**Email:** [sales@sidhil.com](mailto:sales@sidhil.com) / [www.sidhil.com](http://www.sidhil.com)