

SURGI-GEL™ STERILE SACHETS 3mL

Composition	A viscous, water based, sterile adhering gel. Surgi-gel™ Sterile sachets contain purified water, glycerol, carboxymethylcellulose sodium, propyl hydroxybenzoate and methyl hydroxybenzoate as preservatives. Sterilised by gamma irradiation.
Use	Surgi-gel™ Sterile sachets contain a sterile, greaseless, transparent, water-based, non-staining, non-irritating lubricating gel. Recommended for general gynaecological, surgical, urological use and when vaginal lubrication is needed.
Contraindications	Known hypersensitivity to propyl hydroxybenzoate and methyl hydroxybenzoate.
Directions for use	Apply to instruments or directly to mucosal surfaces and orifices as required. Single use only.
Adverse Reactions	<p>Surgi-gel™ Sterile sachets are normally well tolerated, but should skin irritation occur, discontinue use.</p> <p>Sensitisation has occurred when medications containing hydroxybenzoates have been applied to damaged or broken skin.</p> <p>After topical use, local irritation may occur. Hypersensitivity reactions have been reported.</p>
Storage	<p>Store in a cool dry place out of direct sunlight.</p> <p>Not scheduled using the criteria in the Standard Uniform Schedule for Medicines and Poisons.</p>

Presentation

Code	Product	Size
SUR03037F	SURGI-GEL™ STERILE SACHETS	3mL

Contact Point

Medical Information Associate
 Distributed by Perrigo Australia
 Orion Laboratories Pty. Ltd., trading as Perrigo Australia
 ABN 56 009 293 136
 25-29 Delawney Street, Balcatta, Western Australia 6021 AUSTRALIA
 Telephone (all hours): +618 9441 7800
 FREE PHONE: 1800 805 546 FREE FAX: 1800 004 110
 EMAIL: customerservice@perrigo.com.au; WEBSITE: www.perrigo.com.au
 ORION® is a registered trademark of Orion Laboratories Pty Ltd

END OF USER GUIDE