

# Comfeel®

## Plus Transparent

### Effective under masks

### and goggles to protect skin at risk



10x10 BX10 CODE: 29103533

5x7 BX10 CODE: 29103530

#### **This is a challenging time for healthcare professionals.**

The outbreak of the Corona virus is putting immense pressure on both medical personnel and medical facilities around the world. To protect themselves from infection, healthcare professionals are taking additional precautions - using masks and goggles when treating patients. Use of masks and goggles may cause pressure injuries and discomfort due to prolonged wear. Research has shown that hydrocolloids can be used for protection of skin and pressure injuries. Comfeel plus is a soft, comfortable and discrete hydrocolloid dressing that is easy to apply and remove.

#### **APPLICATION VIDEO**

**Watch video for how to apply product.**


[https://youtu.be/TeCTMoVY\\_SQ](https://youtu.be/TeCTMoVY_SQ)

## How to cut Comfeel Plus Transparent

Ensure that the skin is completely dry before applying the product.



Areas where you can experience skin friction: cheek, nose, forehead.

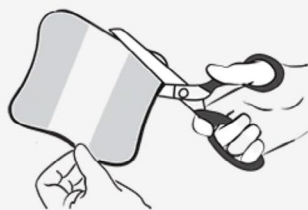

Cut to fit areas where skin is at risk. Remember to cut before removing the protective papers.



Consider aseptic technique during handling and apply with the adhesive side of the product facing the skin. Gently swipe your fingers on the product to make it adhere to the skin.



Put on the protective masks and goggles. A Fit Test may be required depending on the type of mask used. Please refer to and follow your local guidelines.