


TIR-1™

Quick Reference Guide





Components and Features

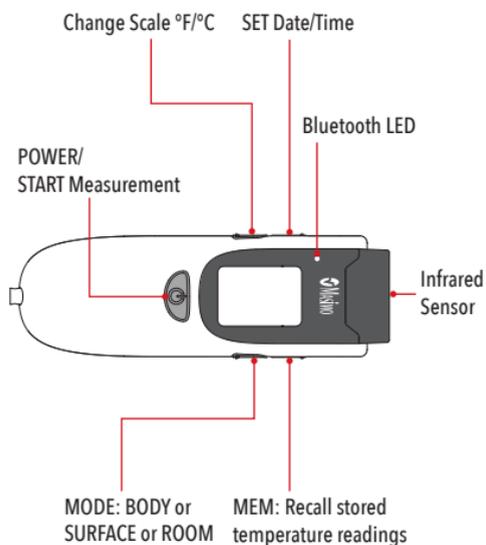

Setup


- 1** Press down on top section of battery cover and slide in the direction of the arrow on the cover.
- 2** Insert 2 AA Alkaline batteries in the compartment and wait for the beep and the LCD to light up.
- 3** Slide battery cover back over the compartment with pressure on both top and bottom of the battery cover.

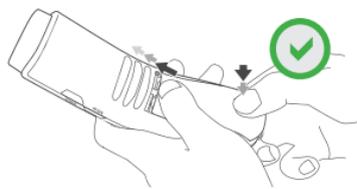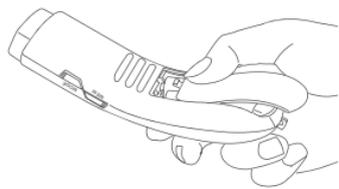

NOTE: Do not place the battery cover over the back compartment and press down with uneven pressure as this can damage the battery door

Bluetooth® Pairing



- 4** Enable the Bluetooth option on the Masimo device
- 5** On the Bluetooth menu, select the Thermometer option
- 6** Power on the TIR-1 unit and take a measurement
- 7** Hold close to the Masimo device and select the unit on the device screen to pair

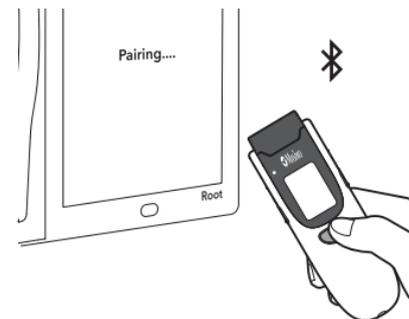

Operation

- 8** Remove protective cap 
- 9** Clean the central area of forehead 
- 10** Aim at middle of forehead, 1/2 - 2 inches away from skin 
- 11** Press and release the POWER button. Hold thermometer in place until 2 beeps are heard and temperature is displayed on LCD.
- 12** View the measurement on the Masimo device screen as well as on the TIR-1 unit

Settings

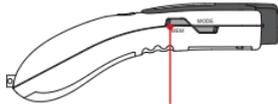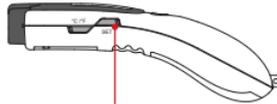Mode Change	Scale Change	Memory	Setting Date/Time
 <p>Mode Button</p>	 <p>Temperature Scale Button</p>	 <p>Memory Button</p>	 <p>Date/Time Button</p>
Steps To Manage Settings			
<ul style="list-style-type: none"> > Press and hold button for 5 sec for current mode > Press button again to cycle to next mode 	<ul style="list-style-type: none"> > Press the button for current scale > Press button again to change scale 	<ul style="list-style-type: none"> > Press MEM button and hold to view most recent measurement ('01') > Release button to view measurement details > Press and hold button again to move to next measurement ('02') 	<p>To View:</p> <ul style="list-style-type: none"> > Press and hold SET button for 5sec for current date and time > Digits flash per section (Month, Day, Year, Hours, Minutes) > To change section press SET button
<p>Available Measurement Modes:</p> <ul style="list-style-type: none"> > Body > Surface > Room 	<p>Available Options:</p> <ul style="list-style-type: none"> > Farenheit > Celsius 	<p>Measurement Details Include:</p> <ul style="list-style-type: none"> > Time > Temperature > Mode 	<p>To Change:</p> <ul style="list-style-type: none"> > Press MODE button to increase > Press °C/°F button to decrease

Not intended to replace the Operator's Manual.

Masimo U.S.
 Tel: 1 877 4 Masimo
 info-america@masimo.com

Masimo International
 Tel: +41 32 720 1111
 info-international@masimo.com

