

Product Description

Lifting very heavy patients requires special techniques and specially designed lift equipment. Liko has developed the Ultra product series to help lift such patients in a safe and comfortable manner.

The Liko Ultra LiftPants also give very heavy patients the chance to practice standing and gait training without risk of falling. They also reduce the risk of injuries to caregivers.

The Liko Ultra LiftPants aid in safe standing and gait training for patients with poor balance and poor leg function. The LiftPants allow the patient to move while relieving him/her of a greater or lesser degree of body weight.

The Liko Ultra LiftPants are equipped with two hip straps on either side for more even weight distribution. There are two different sizes.

Selecting a suitable lift

We recommend that at least two caregivers be present when transferring a patient.

The Liko Ultra LiftPants can be used along with Liko's overhead or mobile lifts. An overhead lift can be used by the patient for unrestricted movement within the lift's lifting area. You can choose between:

- Liko's UltraTwin system with two lift units. The maximum load is 500 kg (1100 lbs.), depending on which individual products are included in the assembled lift system. The UltraTwin system is available for both stationary installation and for freestanding installation in the UltraTwin FreeSpan.
- Viking XL for lifting up to 300 kg (660 lbs.). May be equipped with arm rest to give extra support during standing- and gait training

In this document, the person being lifted is referred to as the patient and the person helping them is referred to as the caregiver.

 is a warning triangle used for situations that demand extra care and attention.


 Read the instruction guide for both the sling and the lift used. Instruction guides are available for downloading, free of charge, at www.liko.com.

Table of Contents

Safety Instructions	3
Care and Maintenance	3
Definitions	4
Applying the LiftPants.....	4-5
Adjustment of Loop Straps	6
Removing the LiftPants	6
Choosing the right size.....	7
Accessories	7
Overview, Liko Ultra LiftPants, Mod. 920.....	8
Recommended combinations.....	8

Safety Instructions

Before lifting, remember the following:

- A responsible person within your organization should decide on a case-by-case basis whether one or more caregivers are needed.
- Ensure that the sling selected meets the patient's needs with reference to model, size, fabric and design.
- For safety and hygiene reasons, use individual slings.
- Before lifting, plan the lifting operation so that it can be done as safely and smoothly as possible.
- Before lifting, check that the lifting accessory hangs vertically and can move freely.
- Although Liko's sling bars are equipped with latches, special caution must be exercised: Before the patient is lifted from the underlying surface, but after the straps have been fully extended, make sure the straps are properly connected to the sling bar.
- Never lift a patient higher off the underlying surface than is needed to complete the lifting and transfer procedure.
- Never leave a patient unattended during a lifting situation.
- Make sure that the wheels on the wheelchair, bed, gurney, etc., are locked during the lifting/transfer operation.
- Always work ergonomically.

Medical technical class I product

Liko Ultra LiftPants have been tested by an accredited testing institute and comply with all requirements for MDD Class I products and the harmonized EN ISO 10535 standard.

Protected by Patent.

Liko is quality certified according to ISO 9001 and its equivalence for the medical device industry, ISO 13485. Liko is also certified according to environmental standard ISO 14001.

Liko's products undergo continuous development, which is why we reserve the right to make product changes without prior notice. Contact your Liko representative for advice and information about product upgrades.

IMPORTANT!

Lifting and transferring a patient always involves a certain level of risk. Read the instruction guide for both the patient lift and lifting accessories before use. It is important to completely understand the contents of the instruction guide. The equipment should only be used by trained personnel. Ensure that the lifting accessories are suitable for the lift used. Exercise care and caution during use. As a caregiver, you are always responsible for the patient's safety. You must be aware of the patient's ability to make it through the lifting situation.

Care and Maintenance

Inspect the sling regularly, especially after washing. Inspect the following points with regard to wear and damage:

- Fabric
- Straps
- Seams
- Suspension loops
- Buckles

Do not use damaged lifting accessories.

If anything is unclear, please contact the manufacturer or supplier.

Washing instructions: See the sling's product label.


Expected Life Time

The product has an expected life time of 1-3 years with normal use. Expected life time varies depending on the fabric, use frequency, washing procedure, and weight carried.

Periodic Inspection


The product must be inspected at least once every 6 months. More frequent inspections might be required if the product is used or washed more often than normal. See Liko's protocol Periodic Inspection for the respective product.

Definitions

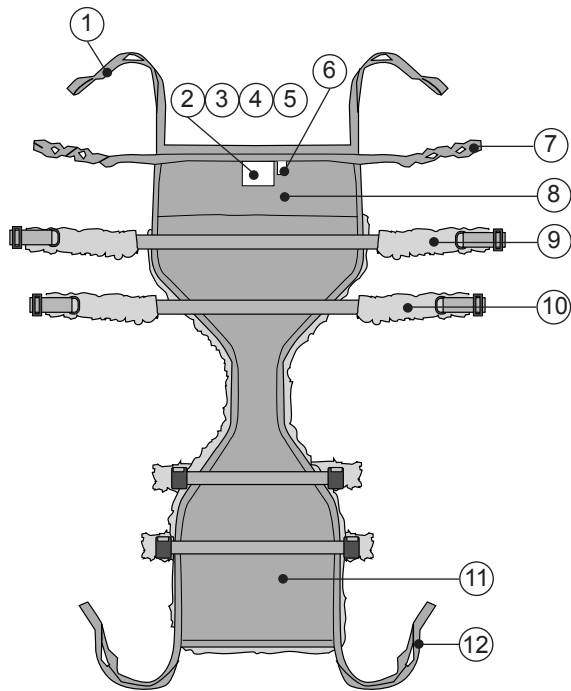


1. Rear suspension straps
2. Product label
3. Serial number
4. Label: Periodic Inspection
5. Label: Individual marking (Belongs to)
6. Size marking (only for size XXL)
7. Loop straps
8. Rear part
9. Upper hip strap
10. Lower hip strap
11. Front part
12. Front suspension strap


Applying the LiftPants

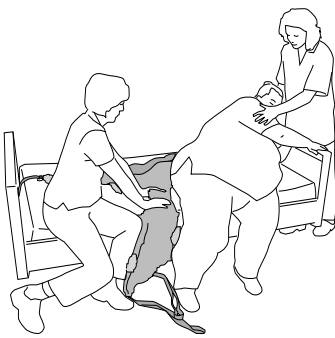

The initial position when using the Liko Ultra LiftPants are the sitting position. The patient leans to the side so that the LiftPants can be slid into place under the buttocks. Place the LiftPants as far under the patient as possible with the product label facing out from the back. Make sure that the LiftPants are centered and in a good position with as much pant material in back as in front. Adjust as needed.

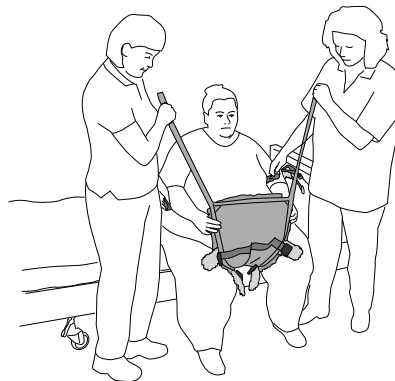


Pull the front section of the LiftPants up between the legs so that the hip straps are accessible from the front.

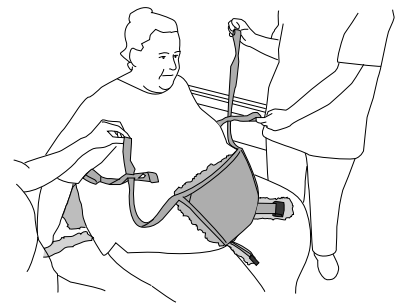


Feed the front suspension straps through the loop straps using the appropriate loop (see page 6, Adjusting the Loop Straps).

Alternative application of the Liko Ultra LiftPants: The patient can be lifted in a normal sling and then lowered onto the LiftPants placed on a chair or at the edge of the bed. The sling used can be e.g. the Liko UltraSling Mod. 06, Liko Original HighBack Sling Mod. 20 or Liko Soft Original HighBack Sling Mod. 26. This application makes it easier if for some reason the patient cannot lean to the side, or if there is no room to apply the LiftPants from the side.

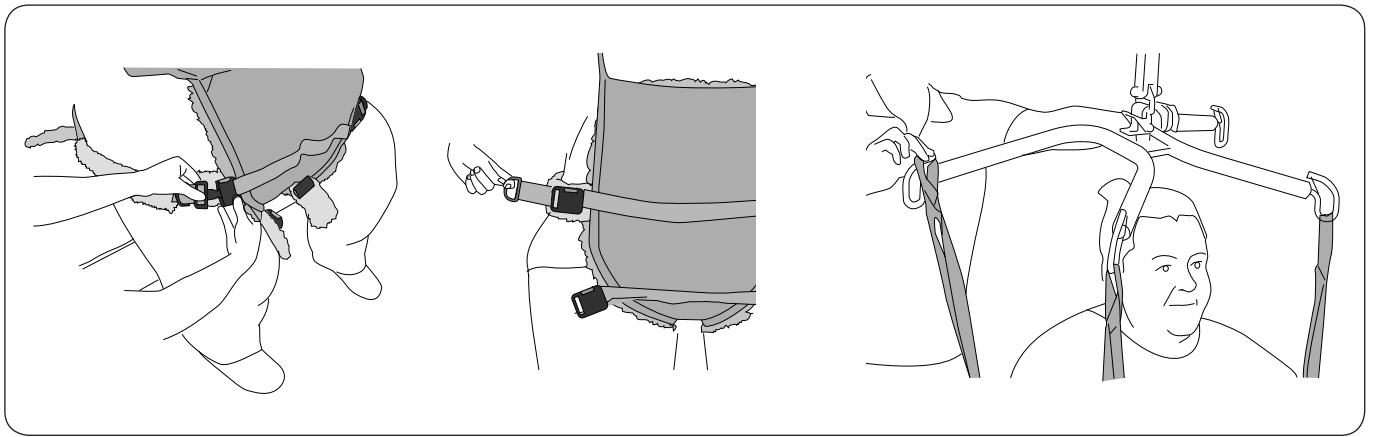

Fasten the hip strap's buckles and pull tight. Depending on the size of the patient, either one or both hip straps may be fastened from the standing position.

Connect the rear suspension straps onto the rear hooks on the sling bar, or onto the rear sling bar. Connect the front suspension straps to the front hooks on the sling bar or to the front sling bar. The outermost strap loop is the most commonly used.

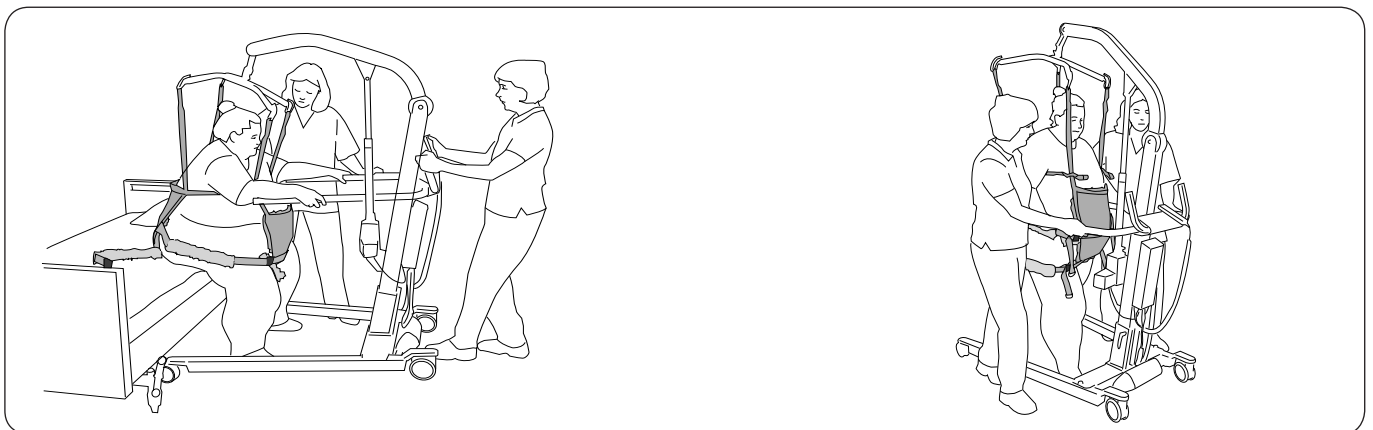


Before assisting the patient to his/her feet: Make sure the patient has a steady footing on the floor to lighten the load during the raising. Run the lift slightly upwards to apply tension to the suspension straps without lifting the buttocks, and make sure that the loop straps are providing adequate support around the patient's upper body.


During the lifting operation: Pull the lift forward in the direction of movement so that weight transfer takes place properly until the patient is upright and well-balanced.

When the patient has gained his/her balance standing, the lifting height is adjusted to provide appropriate relief with help from the LiftPants. Gait training can now begin!


Adjusting the Loop Straps

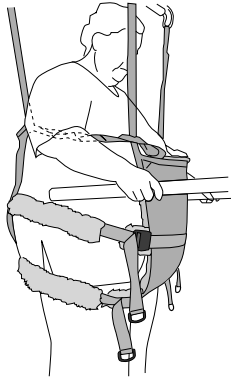

The loop straps should be neither too loose nor too tight around the upper body. Resistance with comfort is best.

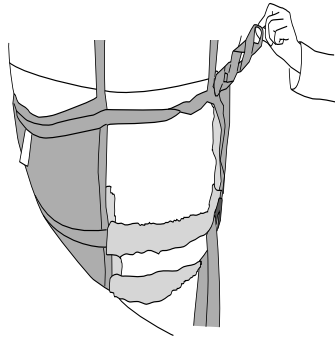


A smaller person can have the suspension straps fed through an inner loop.

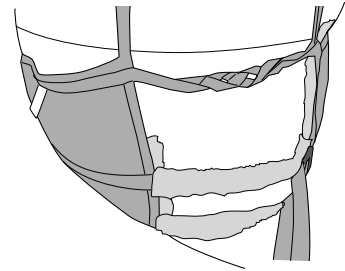


A larger person can have the suspension straps fed through an outer loop.

Loop Clips


A practical accessory for indicating the loops that work for a certain person. This makes it easier for caregivers to choose the right loop when applying the LiftPants.

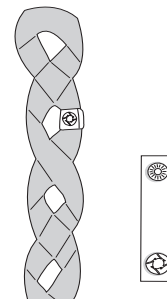

Removing the LiftPants

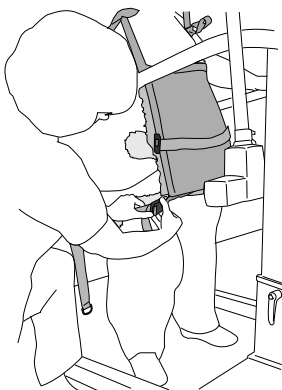

Loosen the hip straps before lowering the patient down into a sitting position to avoid undue pressure from the hip straps.

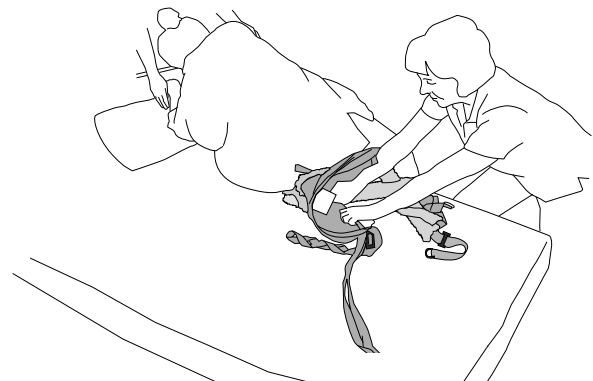


The patient leans to the side. Seize hold of the outer corner of the rear part of the LiftPants. Now fold the LiftPants in under itself and pull it away.

Choosing the right size


Choosing the right size is important for a safe and secure lift and for reaching the required lift height. Liko Ultra LiftPants should reach above waist height but not so high that the loop straps cut into the armpits. Refer to the measurement table below for guidance. Size XXXL has higher fabric sections and longer hip straps in order to reach around a larger belly.

△ Do not choose too large size as this can mean that the required lift height cannot be achieved.

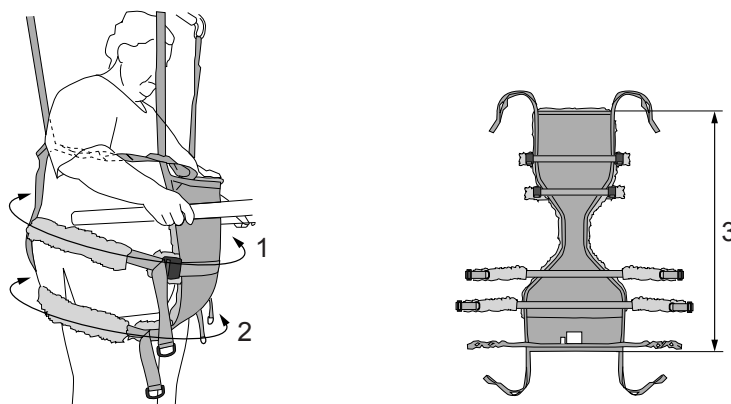

Measurement	1	2	3
XXL	185-260 cm (73-102 inch.)	150-210 cm (59-83 inch.)	140 cm (55 inch.)
XXXL	185-300 cm (73-118 inch.)	150-250 cm (59-98 inch.)	150 cm (59 inch.)

Accessories

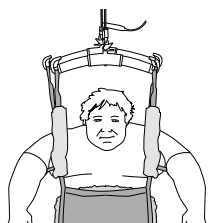


Padded cases for shoulder strap

Prod. No. 3666021

Pressure-relieving padded cases for mounting onto the forward and/or rear suspension straps.

The padded cases come with Velcro along one longitudinal side for easy mounting.

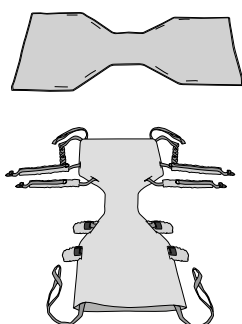

Solo Cover for Ultra LiftPants (XXL/XXXL)

Prod. No. 3692908

Solo Cover for Ultra LiftPants (XXL and XXXL) is a protective cover made from disposable material. It is used to avoid dirtying the LiftPants and to prevent contagion.

Overview, Liko Ultra LiftPants, Mod. 920

Name	Prod. No.	Size	Patient weight ¹	Patient height ¹	Maximum load ²	Material
Liko Ultra LiftPants	35920328	XXL	200-500 kg (440-1100 lbs.)	170-210 cm (67-83 inch.)	500 kg (1100 lbs.)	Net polyester
Liko Ultra LiftPants	35920329	XXXL	200-500 kg (440-1100 lbs.)	170-210 cm (67-83 inch.)	500 kg (1100 lbs.)	Net polyester

¹ The indicated patient weight and height are only guidelines – there may be deviations.

² For retained maximum load, lift/accessories intended for the same load or more are required.

To choose the right size – refer to "Choosing the right size" on page 7.

Recommended combinations

When using one lift system (Likorall 243 and 250, Viking L and XL)

Recommended combinations of the Liko Ultra LiftPants with Liko's sling bars:	Mini 220	Slim 350	Universal 350	Standard 450	Universal 450	Universal 600	Wide 670	Univ. Twinbar 670	Ultra Slingbar	Cross-bar	Cross-bar 450	Cross-bar 670	SideBars + Slim 350	Univ. SideBars + Universal 350	SideBars + Standard 450	Univ. SideBars + Universal 450
Ultra LiftPants, XXL	3	2	3	2	2	1	1	1	4	4	1	1	4	4	4	4
Ultra LiftPants, XXXL	3	3	3	2	2	1	1	1	4	4	1	1	4	4	4	4

Explanation: 1 = Recommended, 2 = May work, 3 = Not advised, 4 = Inappropriate

When using two lift units (UltraTwin), only the below combinations are possible

Recommended combinations of the Liko Ultra LiftPants with Liko's sling bars:	Mini 220	Slim 350	Universal 350	Standard 450	Universal 450	Universal 600	Ultra SlingBar
Ultra LiftPants, XXL	3	3	2	1	1	1	1
Ultra LiftPants, XXXL	3	3	2	1	1	1	1

Explanation: 1 = Recommended, 2 = May work, 3 = Not advised, 4 = Inappropriate

Other Combinations

Combinations of accessories/products other than those recommended by Liko can result in risks for the safety of the patient.

In those cases where another manufacturer recommends Liko's accessories/products in combination with their or others' accessories/products, and the combinations are not approved by Liko, that manufacturer assumes the responsibility for that combination. Liko then refers to the responsible manufacturer's instruction guides and recommendations concerning usage, maintenance, cleaning, checking and inspection of the combination in order to fulfil the EN ISO 10535 requirements.



A **Hill-Rom** Company

www.liko.com

Manufacturer:

Liko AB
SE-975 92 Luleå
Sweden
info@liko.se

